XYLAZINE TEST STRIPS

QUICK REFERENCE GUIDE

This guide shows how to test powdered opioids. See the full instructions for testing pressed opioid tablets.





over 5 ml of water




A measuring cup that holds at least 5 ml of water

10 mg micro scoop

A WHPM xylazine test strip

1 CRUSH & MIX

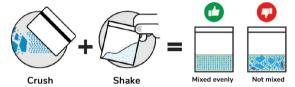

SHOULD ONLY **BE USED TO TEST OPIOIDS**

XYLAZINE STRIPS

CARRY NALOXONE.

When you can, avoid using alone. NeverUseAlone.com 877-696-1996

2. MEASURE & DILUTE























to one teaspoon of water (5 ml)



3. TEST & INTERPRET RESULTS







Lav On Flat Surface Wait 3 Minutes

XYLAZINE

NO XYLAZINE

Any second line, no matter how faint, is still a negative result.

Use a bright